

# Septoject

**Sterile siliconised dental needles with bevel marking**

CE 0459

## INSTRUCTIONS FOR USE

- Break the sterility guarantee seal just before use by turning the two parts of the protective sleeve in opposite directions.
- Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
- Remove the plastic cap just before the injection is administered.  
It can be used again to remove the needle from the syringe to avoid risk of needle stick injury.
- Discard the used needle into a puncture-resistant container in accordance with all applicable medical waste regulations.

## WARNINGS

- Never recap needles by hand. Use only accepted dental recapping procedures. Recapping needles by hand or recapping bent or broken needles can cause unintended needle sticks.
- Do not bend, break, or otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury. Do not use short needles (< 30 mm) when the expected depth in soft tissue approaches the length of the needle.
- Avoid extreme pressure and excessive needle movement during injection as this may cause needles to break, which may result in serious injury to you and/or your patient.
- Be especially vigilant when repositioning a needle in a patient who appears to be apprehensive.
- For Septoject XL: avoid either too rapid of an injection and/or excessive pressure as that may lead to increased patient pain during injection.

## CAUTIONS

- Store in original packaging until ready to use.
- Read the Instructions For Use supplied with your injection device.
- If seal is broken, the needle is NOT STERILE and MUST NOT BE USED.
- For single use only: This device is for use in a single patient. Any reuse exposes other patients and the user to critical risks, including infection and trauma since the needle is no longer sterile and is more likely to break."
- If multiple injections are required, replace the needle for each cartridge in order to minimize the risk of needle point damage.  
1 patient / 1 cartridge / 1 needle
- Advice for first time users of Septoject XL needles: Because XL needles have a larger lumen (bore) than other needles, prior to injection, soft tissue should be stabilized to reduce the risk of needle deflection.

## PRESENTATION

100 two-point disposable non-resterilisable dental needles

***For professional dental use only***

(CE) : 12/1998

Rév. : 12/2013

STERILE R



SEPTODONT

58, rue du Pont de Créteil

94107 Saint-Maur-des-Fossés Cedex - France

Tel.: 33 (0)1 49 76 70 00

