

All-New

# CARRIERE® | M-SERIES™ WIRES

## A Revolutionary 3-Wire Sequence


Introducing the All-New *M-Series*™ Wires, including .015 Cu™ Nitanium® Wire and the innovative 3-Wire Sequence. *M-Series* Wires are designed to:

- Enable a simplified wire progression protocol
- Reduce the number of wires used
- Finish cases in about 12 months

Integral to the *SAGITTAL FIRST*™ Treatment Philosophy is the use of the *Carriere*® *MOTION 3D*™ Appliance to correct AP discrepancies and stimulate initial tooth movement. This fosters easier transitions into larger wires earlier in treatment, less time in braces, and shorter overall treatment times.



**SLX 3D™**  
BRACKET

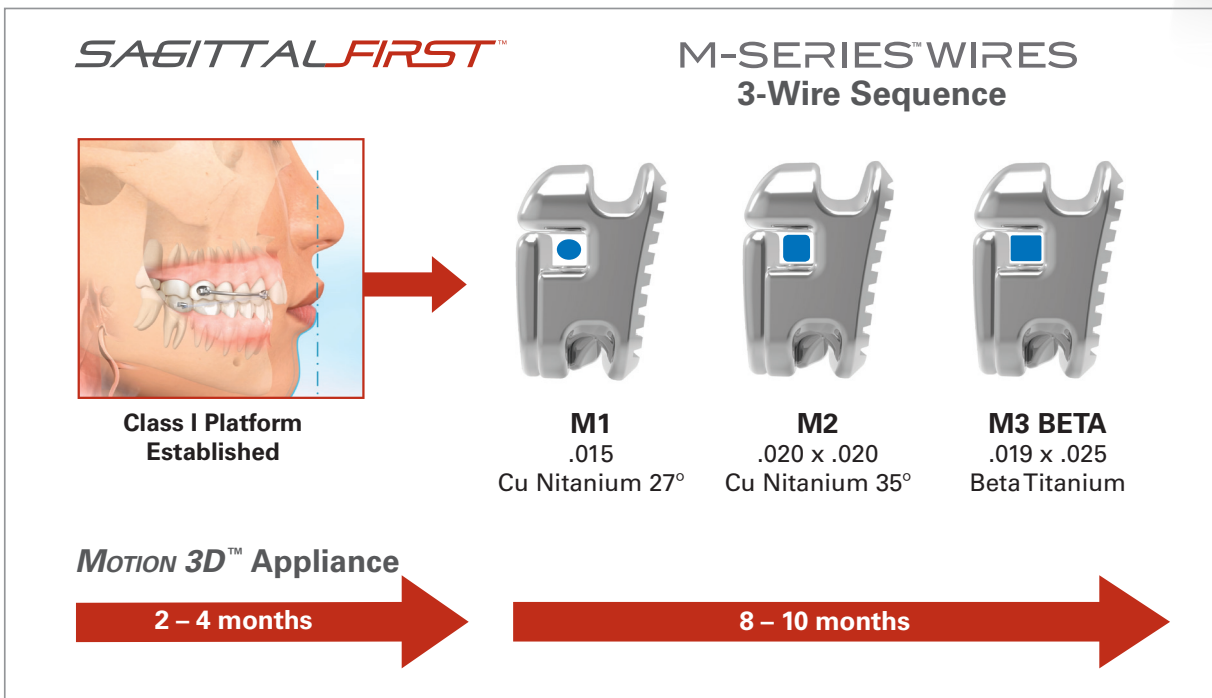

**CARRIERE®**  
M-SERIES™ WIRES

**HENRY SCHEIN®**  
ORTHODONTICS

The New Movement In Orthodontics™

# M-SERIES™ WIRES

## Wire Sequencing Protocol

### M-Series Wire Sequence - for Metal Bracket Systems

	Wire Name	Description	Objective	Time in Treatment	Number of Appointments (5-6 week intervals)	Item Number
<b>NEW SIZE</b>	1 M-ONE	.015 Cu Nitanium 27° with EZ Stops™	Begin leveling, aligning, and rotations	5-6 weeks	1 appointment	103-M1
	2 M-TWO	.020 x .020 Cu Nitanium 35° with EZ Stops	Complete leveling, aligning, and elimination of rotations. Arch development is initiated, 3 dimensional control of wire will begin initial torque control	6-18 weeks, based on complexity and clinical judgement	2-3 appointments	103-M2
	3 M-THREE BETA	.019 x .025 Beta Titanium	Detailing and finishing	8-12 weeks	2-3 appointments	103-M3B

### Optional Wires

Wire Name	Description	Objective	Time in Treatment	Number of Appointments (5-6 week intervals)	Item Number
M-ONE.FIVE	.018 Cu Nitanium 27° with EZ Stops	Used in small number of cases for stubborn rotations prior to placing M-TWO wire	4-6 weeks	1-2 appointments	103-M1.5
M-TWO.FIVE	.019 x .025 Cu Nitanium 35° with EZ Stops	For final bracket alignment prior to M-THREE BETA wire with ceramic brackets	3-4 weeks	1 appointment	103-M2.5
M-THREE STEEL	.019 x .025 Stainless Steel	Surgical stabilization wire placed prior to surgery and/or to complete expansion in stubborn cases; used instead of M-THREE BETA	8-12 weeks	2-3 appointments	103-M3S

All M-Series Wires are available in the D-LX™ archform and are packaged in single pouches in packs of 10.

Visit [CarriereSystem.com](http://CarriereSystem.com) for more information.



The New Movement In Orthodontics™

For more information on our products and educational offerings, please contact us:

HenryScheinOrtho.com | Customer Service: 888.851.0533

In the U.S.: 800.547.2000

Outside the U.S.: +(1) 760.448.8600

Canada: CERUM 800.661.9567

To email an order: [USASales@HenryScheinOrtho.com](mailto:USASales@HenryScheinOrtho.com)

To email an order from outside the U.S.: [IntlOrders@HenryScheinOrtho.com](mailto:IntlOrders@HenryScheinOrtho.com)

Ortho Organizers, Inc. 1822 Aston Ave. Carlsbad, CA 92008-7306



**CARRIERE**  
Shifting the way you think about orthodontics.