EN FR DE IT ES TR

Fantasista V

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- For the best handling, use Fantasista V at room temperature.
- If the cavity preparation is large and/or deep, incremental filling techniques should be employed.
- If dispensed material is not used immediately, cover it to protect the material from ambient light preventing premature polymerization.

EN

INSTRUCTIONS

IMPORTANT :
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Resin-based Dental Restorative Material

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Fantasista V is a visible light cure, tooth shade, radio-opaque composite resin using an organic filler which is reactive with the polymer matrix. The inorganic filler loading is 53.0 vol% with a particle size range of 0.02 to 5.0 µm.

Indications :

Direct anterior and posterior restorations including the occlusal surface and all carious classes.

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Composition

Methacryate monomers, Organic filler, Inorganic filler, Aromatic amine, etc.

Precautions

- This product contains substances that may cause allergic reactions.
- Fantasista V should not be used either by clinicians or on patients who are methacrylate monomer-sensitive.
- Avoid contact of Fantasista V with soft tissues, skin or eyes. Contaminated skin or mucosa with Fantasista V should be wiped off immediately with alcohol and then rinsed thoroughly with running water. If Fantasista V enters an eye, rinse the eye with running water immediately. The patients should then be examined by an ophthalmologist.
- Dentist should wear rubber or PVC dental gloves.
- If necessary, the pulp must be indirectly capped locally for deep cavities with only a minimal layer of dentin.
- Avoid accidental ingestion while applying and rinsing this material. If ingested, the patient must seek medical attention.
- Use a dental rubber dam to isolate the treatment area from contamination.
- Use Fantasista V only for the applications recommended in this leaflet.

Storage

Store Fantasista V at a room temperature between 1-30°C. Avoid high temperatures above 30°C, high humidity and direct sunlight. Refrigeration is not necessary. However, if stored in a refrigerator, then allow the material to warm to room temperature prior to use.

Shelf life: 3 years in a room temperature. See outer package for expiration date. Do not use after expiration date.

Disposal

Dispose of empty containers, etc. in accordance with local regulations.

How to use Fantasista V

Adhesive Bonding System :

Apply light-cured bonding system according to manufacturer’s instructions. We recommend the use of Hybrid Bond from Sun Medical Co., Ltd.

Placement and Curing :

- Dispense Fantasista V on to a mixing pad and promptly recap the syringe.
- Place dispensed material into prepared cavity previously treated with the bonding agent.
- Light-cure using dental visible light-curing device according to the manufacturer’s instructions.

Type	Curing time (sec)	Depth of cure (mm)		
		A1	A2, A3, A3.5, A4, BU	A2O, A3O
Conventional halogen	20	2.5	2.0	1.5
Fast halogen	10	2.0	1.5	1.5
	20	2.5	2.0	2.0
Plasma arc	3	2.0	1.5	1.5
	6	2.5	2.0	2.0
LED	10	2.0	1.5	1.5
	20	2.5	2.0	2.0

- For the best handling, use Fantasista V at room temperature.
- If the cavity preparation is large and/or deep, incremental filling techniques should be employed.
- If dispensed material is not used immediately, cover it to protect the material from ambient light preventing premature polymerization.

Polishing :

Fantasista V will polish quickly and easily to a high luster. Use of most discs or polishing paste systems are acceptable.

As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.

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