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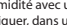
EN  **Instructions for use**
MM-MTA™ – MM-MTA™ Caps Endodontic Repair Cement
MM-MTA™ is an endodontic repair cement. MM-MTA™ powder consists of very fine hydrophilic particles of several mineral oxides. After contact with the liquid, the MM-MTA™ forms a gel that hardens to an impermeable barrier. MM-MTA™ is delivered in MM-MTA™ Caps. MM-MTA™ Caps are quickly and easily activated and the contents of the capsule are easily ejected with the applicator.

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Capsule mixing (mixing time 30 seconds) is achieved using a high frequency mixer that delivers about 4,300 oscillations/min.

- I. INTENDED USES**
- Repair of root perforations during root canal treatment.
 - Root end filling (retrograde)
 - Pulp capping.
 - Root end filling (orthograde)

II. CONTRAINDICATIONS
Not known.

III. SIDE EFFECTS
Not known.

IV. ACTIVATION OF THE CAPSULE AND MIXING
See Instruction for MM-MTA™ Caps)
Activate and mix the the MM-MTA™ Caps according to the information in the MM-MTA™ Caps instruction. Mixing time for the MM-MTA™ Caps is 30 seconds.

Attention:
Avoid lag times between the activation, mixing and application processes as the material is in the process of setting which may impair or prevent application of the material. The material must be extruded within 10 seconds after the end of the mixing process. To prevent dehydration during setting, apply MM-MTA™ intraorally immediately after mixing. Working time of MM-MTA™ is approx. 2 minutes (at 23°C).

V. APPLICATION
5.1. Repair of root perforations
Place rubber dam and clean the root canal system using intra-canal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation. Fill the apical canal space up to the perforation completely with a suitable root canal filling material. Mix MM-MTA™ as described in paragraph 4 and extrude it on a clean glass plate. Apply MM-MTA™ with suitable instruments into the perforation site and condense it. Check the position of MM-MTA™ in the root canal using an X-ray. If the material does not form an even layer, rinse MM-MTA™ out of the canal and repeat the procedure. Remove excess moisture with a damp cotton pellet or a paper point. Place a damp cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the access preparation with a suitable root canal filling material and seal the cavity with a tight filling. Both options may only be carried out 5 minutes after placement of the MTA-Universal. MTA-Universal repair material remains a permanent part of the root canal filling.

5.2. Root End Filling (retrograde)
Create an access to the root-end and resect the root with a surgical bur. Use an ultrasonic tip to prepare a class I root-end cavity preparation to a depth of 3-5 mm. Isolate the area and dry the root end cavity with paper points. Carry out hemostasis with suitable methods. Mix MM-MTA™ as described in paragraph 4 and extrude it on a clean glass plate. Apply MM-MTA™ with suitable instruments and condense it using a small plugger. Remove excess cement and clean the surface of the root with a moist piece of gauze. Confirm placement of the MM-MTA™ repair material by means of an X-ray. The MM-MTA™ repair material becomes a permanent part of the root canal filling.

5.3. Pulp Capping
Place rubber dam and prepare the access cavity. If there is decay, remove it. Rinse cavity and exposed pulpal area with a suitable disinfectant. Mix MM-MTA™ as described in paragraph 4 and extrude it on a clean glass plate. With a suitable instrument apply a small amount of MM-MTA™ over the exposed pulp and remove excess moisture with a dry cotton pellet. Wait at least 5 minutes after applying MM-MTA™ before placing a small amount of a flowable light cure liner and light cure. Etch the remaining cavity walls with gel according to the total-etch technique and apply a suitable bonding agent according to the corresponding instructions.

To finish the restoration, place a light cure composite cement carefully. Pulp vitality and status should be checked using X-rays at regular intervals.

5.4. Root End Filling (orthograde)
Place rubber dam and clean the root canal system using intra-canal instruments and irrigate with NaOCl. Dry the root canal with paper points. For disinfection place calcium hydroxide paste in the root canal for one week. Seal the access opening with a temporary filling material. Mix MM-MTA™ as described in paragraph 4 and extrude it on a glass plate. With a suitable instrument, apply a small amount of MM-MTA™ into the apical region and condense it. Create a 3 – 5 mm barrier of MM-MTA. Check the position of MM-MTA™ with an X-ray. If the material does not form an even layer, rinse MM-MTA out of the canal and repeat the procedure. Remove excess moisture with a damp cotton pellet or a paper point. Place a damp cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the access preparation with a suitable root canal filling material and seal the cavity with a tight filling. Both options may only be carried out 5 minutes after placement of the MM-MTA. MM-MTA repair material becomes a permanent part of the root canal filling.

VI. PRECAUTIONS FOR USE
In the first hour after application handle the placed MTA cement carefully.
• Store MM-MTA™ in the sealed packaging in a dry place until it to be used.
• Intraoral application of MM-MTA™ must be done immediately after mixing to prevent dehydration during setting.
• MM-MTA™ can cause discolouration. Use MM-MTA™ only in the root canal and/or the pulp chamber.

VII. STORAGE
Store MM-MTA™ in a dry place at 10 – 25°C. Do not store below 10°C!
Do not use after expiry date.
MM-MTA™ Caps are for single use only.

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VIII. WARRANTY
S&C Polymer Silicon- und Composite Spezialitäten GmbH warrants this product will be free of material and manufacturing defects. MICRO-MEGA® disclaims all other liabilities and does not implicitly warrant that it is fit for commercialisation or adaptable to a particular purpose. The user shall be liable for determining the suitability of the product for the user's application. In the event of a fault with the product during the warranty period, the sole obligation of S&C Polymer shall be to repair or replace the product of S&C Polymer Silicon- und Composite Spezialitäten GmbH.

IX. LIMITATION OF LIABILITY
Except where prohibited by law, S&C Polymer Silicon- und Composite Spezialitäten GmbH disclaims all liability for any loss or damage arising from this product, whether direct, indirect, specific, incidental or consequential, regardless of the legal grounds, including warranty, contract, negligence or premeditation.

Keep away from children!
For dental use only!
Caution: US Federal law restricts this device to the sale by or on the order of a dentist.
Do not use if package is damaged

Instructions for activating and mixing MM-MTA™ Caps
a. Powder
b. Liquid
c. Plunger

1. MM-MTA™ Cap before activation.
2. For activation of the MM-MTA™ Cap press the plunger on a hard and flat surface until it penetrates the MM-MTA™ Cap.
3. Click once to activate. Insert the MM-MTA™ Cap into the applicator and click once to activate. Note: The plunger must level with the rear of the capsule.
4. Insert the MM-MTA™ Cap into a mixer (or an amalgamator), close lid and mix immediately for 30 seconds (about 4300 oscillations / min).
5. Remove the cap from the capsule. If this is not done, the capsule can burst.
6. Insert the MM-MTA™ Cap into the applicator. Pull the lever 2 times (2 clicks) to prime the MM-MTA™ Cap. Extrude the contents of the capsule on a glass plate. Unlock the applicator (push button A) and remove the MM-MTA™ Cap. The optimal amount of mixed material can only be guaranteed if the applicator is used.

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